Insertion Cannula for UreSil® THORA-VENT® Thoracic Vent 11F, 13cm

en

INTENDED USE:

This over-the-wire cannula is intended for Seldinger Introduction of UreSil THORA-VENT Thoracic vent - 11F x 13cm. TV11-13

This product is intended for transient use (less than 24 hours).

INSTRUCTIONS FOR USE:

This package contains instructions for the specific use of the over-the-guidewire cannula. For additional instructions related to the use of the Thoracic Vent, see the instructions for use that are included in the Thoracic Vent Procedure Trays.

1. Select and prepare the site as described in the instructions provided in the Thoracic Vent Procedure Tray.
2. Using the scalpel, make a small incision at the selected site.
3. Using Seldinger technique, dilate the tract to approximately one French size larger than the catheter on the Thoracic Vent.
4. Remove the protective tube from the cannula and insert it through the self-sealing port. Lock the cannula into the Thoracic Vent.

NOTE: The grooved Thoracic Vent trocar is not used when introducing the device using the Seldinger technique.

5. Peel away the center portions of the paper cover from the adhesive patch on the Thoracic Vent.
6. Advance the distal end of the catheter/cannula assembly of the Thoracic Vent over the guidewire and into the pleural space. When the tip of the catheter enters the pleural space: stop advancing, unlock the cannula and introduce the full length of the catheter. Remove the cannula and the guidewire. Do not re-insert the cannula unless the Thoracic Vent has been completely removed from the patient.
7. Read the instructions for use in the Thoracic Vent Procedure Tray for additional information.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians. UreSil and THORA-VENT are registered trademarks of UreSil, LLC.

U.S. Patent no. 6,770,063.

TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106
www.uresil.com

Manufactured by:
UreSil, LLC,
5418 W. Touhy Ave., Skokie, IL 60077 U.S.A.
Authorized Representative in European Community:
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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