
(1)


(2)

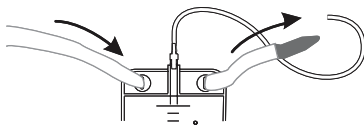(3)

(4)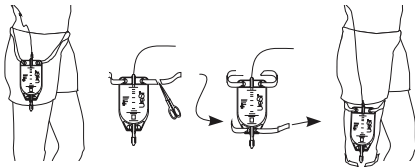

Sterilized using ethylene oxide For single use only

Manufactured By: UreSil, LLC
5418 W. Touhy Ave., Skokie, IL
60077 U.S.A.

EC REP Authorized Representative
in European Community:
EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, The Netherlands

To Reorder: Call 1-800-538-7374 (U.S.A. only)
Fax: (847) 982-0106
www.uresil.com

8-19-200 rev. 01/11/2021


UreSil® Optimus® 600 ml Gravity Drainage Bag

EN

DEVICE DESCRIPTION: 600 ml Gravity Drainage Bag

INTENDED USE:

For collecting drainage fluids.
This product is intended for short-term use (more than 24 hours and less than 30 days).

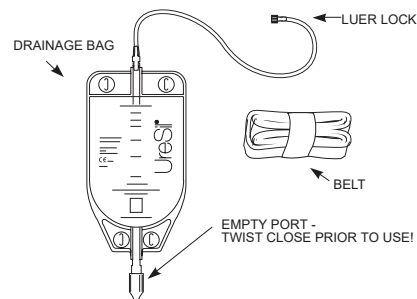NOTE: Graduation marks on bag not intended to measure – for reference only

CAUTION:

Use care if emptying bag to avoid contact with drained fluids.

INSTRUCTIONS FOR USE:

1. Close the port prior to use.
2. Thread the belt as shown.
3. Connect the drainage catheter to the bag tubing as shown.
4. If altering the bag tube length, using sterile technique, remove the luer lock, cut the tubing and re-attach the luer lock.
5. Suspend the bag at a location below the drainage site as shown.
6. Replace bag every 5 to 7 days or sooner if clinically indicated, for example, if malodorous or damaged.

Sterile if package is unopened and undamaged.
The reuse of this single-use device can lead to patient infection and/or device malfunction.

Optimus and UreSil are registered trademarks of UreSil, LLC.

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