
Thoracic Vent Procedure Tr

INTENDED USE: THE THORA-VENT Thoracic Vent Procedure Tr has been designed for the treatment of spontaneous, traumatic or iatrogenic pneumothorax...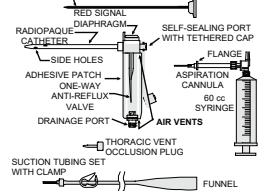

INSTRUCTIONS FOR USE:

- 1. Select a site, preferably at the second intercostal space in the mid-clavicular line or at another appropriate location.
2. Prepare the patient for the procedure. Using the scalpel, make a small skin incision at the selected site.
3. Remove the adhesive patch from the thoracic vent device and introduce the thoracic vent device through the self-sealing port...

CHECK FOR CONTINUOUS AIR LEAK

1. The Thoracic Vent occlusion plug can be used to check for a continuous air leak from the patient's lung. To occlude the thoracic vent device, remove the occlusion plug and place the occlusion plug in place.

ASPIRATION

1. Air can be manually pumped out of the pleural space using the aspiration cannula. Insert the aspiration cannula into the vent through the self-sealing port. Attach the 60 cc syringe to the aspiration cannula using the luer fitting, and utilize the syringe to aspirate air from the pleural space. Do not aspirate directly to the self-sealing port.

EXTERNAL SUCTION ATTACHMENT

1. If attachment to a suction system is desired, remove the suction tubing set from its package. Close the clamp on the tubing set and then open the clamp. Attach the suction tubing set to the self-sealing port on the Thoracic Vent and lock it in place by twisting the luer lock.

WARNINGS/COMPLICATIONS

- 1. Do not rotate the trocar during its introduction through or removal from the thoracic wall on the Thoracic Vent.
2. Do not attempt to reinsert the trocar unless the catheter has been completely removed.
3. DIFFICULT CATHETERIZATION: If the catheter is inadvertently blocked through the rib perforator of the trocar it is re-introduced after placement of the catheter in the chest wall...

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians.

Contains or presents of phthalate bis(2-ethylhexyl) (BBP)

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