


## THORA-VENT®

Treatment of spontaneous, traumatic, or iatrogenic *simple pneumothorax*

Self-contained vent allows for complete ambulation during resolution

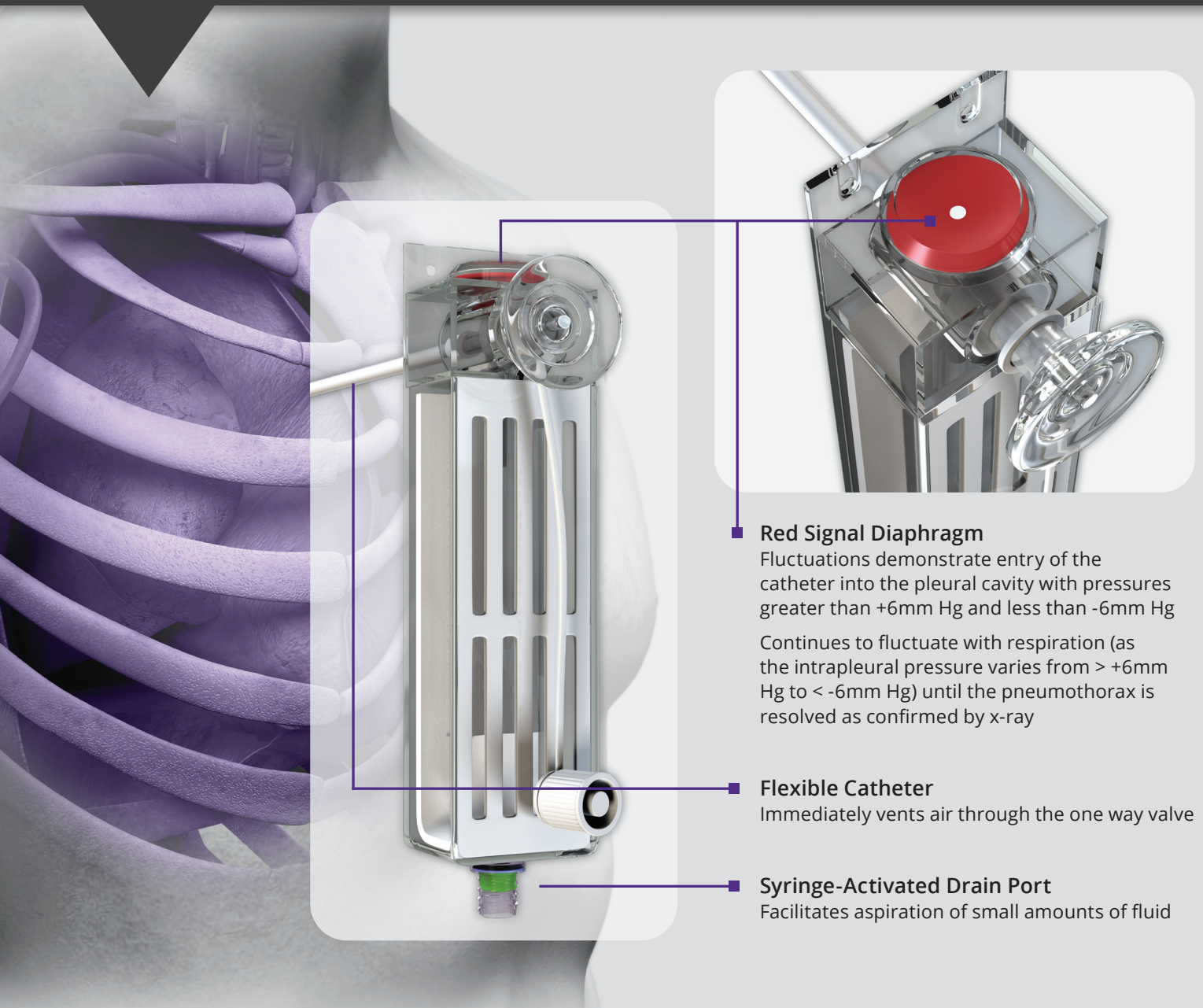

### ■ Red Signal Diaphragm

Fluctuations demonstrate entry of the catheter into the pleural cavity with pressures greater than +6mm Hg and less than -6mm Hg

Continues to fluctuate with respiration (as the intrapleural pressure varies from > +6mm Hg to < -6mm Hg) until the pneumothorax is resolved as confirmed by x-ray

### ■ Flexible Catheter


Immediately vents air through the one way valve

### ■ Syringe-Activated Drain Port


Facilitates aspiration of small amounts of fluid

# THORA-VENT®


## Procedure Steps

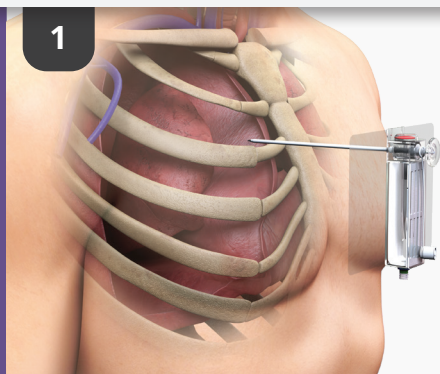

- The THORA-VENT is positioned and gently introduced through a small incision
- The trocar tip perforates the pleural membrane and enters the pleural cavity
- Fluctuation of the red signal diaphragm demonstrates entry of the catheter into the pleural cavity with pressures  $> +6\text{mmHg}$  and  $< -6\text{mmHg}$ , at which point the trocar is not advanced any further

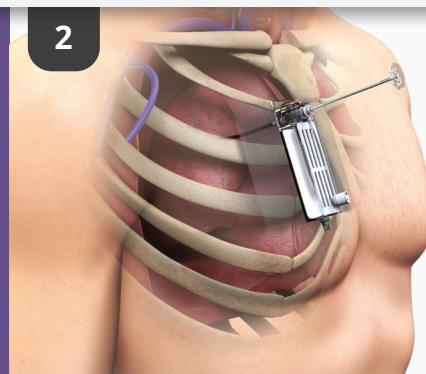

- The full length of the catheter is introduced into the pleural cavity using the trocar as a stabilizing guide
- The trocar is removed

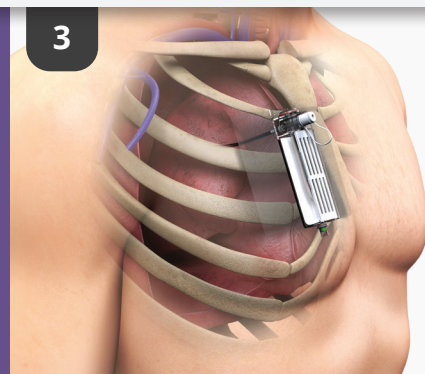

- The device is secured using the included adhesive patch and / or sutures through provided suture holes

## Quick, Convenient Access

Procedure Tray includes components necessary for immediate insertion and evacuation

### Procedure Tray Contains

- THORA-VENT
- Trocar
- Aspiration cannula
- Suction tubing set
- 60cc syringe
- Scalpel
- Halstead forceps
- Gauze sponges (2)
- Fenestrated drape
- CSR wrap
- Occlusion plug



UreSil, LLC

5418 W. Touhy Ave., Skokie, IL 60077

Made in the U.S.A.

Tel: 847.982.0200 Fax: 847.982.0106

Product is CE-marked and available for sale in the USA and in countries where the product has received clearance for sale where applicable.

UreSil and THORA-VENT are registered trademarks of UreSil, LLC.  
U.S. patent no. 6,770,063. MK-TVb Rev. Date 06/15/2021

Ordering Information • 800.538.7374

### THORA-VENT Procedure Tray

Product Codes	Cath. O.D. (Fr.)	Cath. Length (cm)
TV11-10	11	10
TV11-13	11	13
TV13-10	13	10
TV13-13	13	13

### THORA-VENT Accessories

Over-the-Wire Insertion Cannula for TV11-13

Product Code	Units/box
OTW11-13	5

Suction Tubing Set

Product Code	Units/box
TVST	5

Occlusion Plug

Product Code	Units/box
TVOCP	5

  
UreSil®  
UreSil.com