UreSil ® Origin ® Drainage Catheter

INTENDED USE: Percutaneous drainage of abscess, nephrostomy and other fluids. This product is intended for short-term use (less than 30 days).

INTENDED USERS: This product is intended only for use by trained clinicians.

NON- LOCKING CATHETER:

GENERAL PURPOSE CATHETER WITH “J” TIP

HARDWARE:

METAL STIFFENING CANNULA

TROCAR

NOTE: Depth marks (if present) not intended to measure – for reference only

INSTRUCTIONS FOR USE:

1. To maximize advantages of the surface, WET THE CATHETER before use.
2. Straighten the catheter when inserting the cannula (and trocar if appropriate).
3. Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement. Catheters with a trocar included can also be inserted using direct puncture.
4. For pigtail catheters, rotate the catheter clockwise while removing the cannula to form the pigtail.
5. When the catheter is positioned, remove the hardware and fix the catheter in place.

CAUTION:

1. When engaging the catheter luer lock, do not overtighten. Do not use instruments to tighten connections. Overtightening the luer can result in luer crack which may require catheter replacement.
2. Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.

WARNING: The reuse of this single-use device can lead to patient infection and/or device malfunction.

SAFE DISPOSAL: Dispose of used device in container marked for biohazard (i.e. contaminated with potentially infectious substances of human origin).

REPORTING SERIOUS INCIDENTS: Any serious incident that has occurred in relation to the device should be reported to UreSil as the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

URESIL and Origin are registered trademarks of UreSil, LLC. GP is a trademark of UreSil, LLC.

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