DA

Sugeslange til thoraxventil UreSil THORA-VENT

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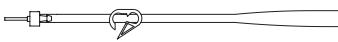
MD

Suction Tubing for UreSil® THORA-VENT®

INTENDED USE: For use with Thoracic Vent if attachment to a suction system is desired. This product is intended for short-term use (less than 30 days).

INTENDED USERS: This product is intended only for use by trained clinicians.

Suction Tubing Set

INSTRUCTIONS FOR USE:

If attachment to a suction system is desired, remove the suction tubing set from the pack. Cleanse the port on the tubing set whilst the tube is being connected. Insert the cannula through the self-sealing port on the Thoracic Vent and lock it in place by twisting the luer lock. Attach the funnel to the suction system and then open the clamp. Appropriate evacuating pressures should be used. Do not exceed a 14.7 mm Hg (20 cm water) vacuum. Always clamp the suction tubing set when suction is not being applied to the Thoracic Vent. If the suction tubing set is no longer required, remove it and immediately seal the self-sealing port with the tethered cap.

Read the instructions for use in the Thoracic Vent Procedure Tray for additional information.

WARNING:

- Always clamp the suction tubing set when suction is not being applied to the Thoracic Vent.
- The reuse of this single-use device can lead to patient infection and/or device malfunction.

SAFE DISPOSAL: Dispose of used device in container marked for biohazardous or contaminated with potentially infectious substances of human origin.

REPORTING SERIOUS INCIDENTS: Any serious incident that has occurred in relation to the device should be reported to UreSil as the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

UreSil and THORA-VENT are registered trademarks of UreSil, LLC. U.S. Patent no. 6,770,063.

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