

Evolution[®] PRE-VAC WOUND DRAINAGE BOTTLE

1000 mL



Refer to the label




Caution: Federal law (U.S.A.) restricts this device to sales by or on the order of a physician.

Re-use of this product will cause the risk of cross infection and unpredictable health threat.

This product is not made with natural rubber latex.

Directions for use

1. Open the pouch.
2. Inspect the vacuum indicator is at "MAX" line. If not, do not use this product.
3. CLOSE CONNECTING TUBE CLAMP FIRST! Attach Male Luer Connector of the connecting tube (if provided) to Female Luer of patient drainage device.
4. After checking all connections are secure, open the white slide clamp on the bottle to activate vacuum.
5. Open connecting tube clamp to begin drainage procedure.
6. Repeat steps 1 to 5 with new bottle if vacuum indicator is at "MIN" Line, or bottle is almost full and further drainage is required.

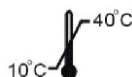

Rx ONLY

Manufactured By : PACIFIC HOSPITAL SUPPLY CO., LTD.

4F., NO. 160, DAYE ROAD, BEITOU DISTRICT, TAIPEI 112, TAIWAN

Factory : No. 8, Tongke 2nd Rd., Tongluo Township, Miaoli 366, Taiwan

Tel : 886 2-2895-5050 Fax : 886 2-2897-8282

E-mail : pahsco@pahsco.com.tw

Made In Taiwan

Distributed by:

URESIL, LLC

5418 W. Touhy Ave.

Skokie, IL 60077

USA



Foreign Exporter:



Canadian Hospital Specialties Ltd.
2810 Coventry Road, Oakville, ON
CANADA, L6H 6R1, www.chsltd.com

MED-RX[®]

TO REORDER:

Call: 1-800-538-7374

Fax: 1-847-982-0106

www.uresil.com

Evolution and UreSil are registered trademarks of UreSil, LLC.

LB010874.F.2021.04