EMERGO EUROPE, Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

UreSil, LLC,
5418 W. Touhy Ave, Skokie, IL,
60077 U.S.A.

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INTENDED USE: This over-the-wire cannula is intended for Seldinger Introduction of UreSil THORA-VENT Thoracic Vent - 11F x 13cm: TV11-13. This product is intended for transient use (less than 24 hours).

INTENDED USERS: This product is intended only for use by trained clinicians.

INSTRUCTIONS FOR USE:
This package contains instructions for the specific use of the over-the-guidewire cannula. For additional instructions related to the use of the Thoracic Vent, see the instructions for use that are included in the Thoracic Vent Procedure Trays.

- Select and prepare the site as described in the instructions provided in the Thoracic Vent Procedure Tray
- Using the scalpel, make a small incision at the selected site.
- Using Seldinger technique, dilate the tract to approximately one French size larger than the catheter on the Thoracic Vent.
- Remove the protective tube from the cannula and insert it through the self-sealing port. Lock the cannula into the Thoracic Vent.
- Advance the distal end of the catheter/cannula assembly of the thoracic vent over the guidewire and into the pleural space. When the tip of the catheter enters the pleural space: stop advancing, unlock the cannula and introduce the full length of the catheter. Remove the cannula and the guidewire. Do not reinsert the cannula unless the Thoracic Vent has been completely removed from the patient.
- Read the instructions for use in the Thoracic Vent Procedure Tray for additional information.

WARNING: The reuse of this single-use device can lead to patient infection and/or device malfunction.

SAFE DISPOSAL: Dispose of used device in container marked for biohazard (i.e. contaminated with potentially infectious substances of human origin).

REPORTING SERIOUS INCIDENTS: Any serious incident that has occurred in relation to the device should be reported to UreSil as the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

UreSil and THORA-VENT are registered trademarks of UreSil, LLC.

EN Catalog number
CS Katalogové číslo
DA Katalognummer
DE Katalognummer
EL Αριθμός καταλόγου
ES Número de catálogo
FI Luettelumero
FR Numéro de catalogue
IT Numero di catalogo
NL Catalogusnummer
NO Katalognummer
PT Referência
SK Katalógové číslo
SV Katalognummer
TR Katalog numarası
ZH 产品目录号

EN Batch code
CS Kód šarže
DA Batchingode
DE Chargencode
EL Αριθμός παρτίδας
ES Código de lote
FI Eräkoodi
FR Code de lot
IT Codice lotto
NL Batchcode
NO Partikode
PT Código do lote
SK Kód šarže
SV Batchkod
TR Parti kodu
ZH 批号

EN Caution
CS Upozornění
DA Forsigtig
DE Achtung
EL Προσοχή
ES Precaución
FI Huomio
FR Avertissement
IT Attenzione
NL Opgelet
NO Advarsel
PT Atenção
SK Upozornenie
SV Varning
TR Dikkat
ZH 注意

EN Authorized representative in the European Community / European Union
CS Způsobilý zástupce v Evropském společenství / Evropské unii
DA Autoriseret repræsentant i Det Europæiske Fællesskab/Den Europæiske Union
DE Bevollmächtigter in der Europäischen Gemeinschaft / Europäischen Union
EL Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Ευρωπαϊκή Ένωση
ES Representante autorizado en la Comunidad Europea / Unión Europea
FI Valtuutettu edustaja Euroopan yhteisössä / Euroopan unionissa
FR Représentant autorisé dans la Communauté européenne et l'Union européenne
IT Rappresentante autorizzato nella Comunità europea/Unione Europea
NL Geautoriseerde vertegenwoordiger in de Europese Gemeenschap / Europese Unie
NO Autorisert representant i Det europeiske fellesskap / Den europeiske union
PT Representante autorizado na Comunidade Europeia / União Europeia
SK Spoločný zástupca v Európskom spoločenstve alebo Európskej únii
SV Auktoriserad representant inom EU/EES
TR Avrupa Topluluğu / Avrupa Birliği/indeki yetkili temsilci
ZH 欧洲共同体/欧盟的授权代表

MedEnvoy, Prinses Margrietplantsoen 33
Suite 123, 2955 AM, The Hague, The Netherlands

EN Importer in the European Community / European Union
CS Dovozce v Evropském společenství / Evropské unii
DA Importør i Det Europæiske Fællesskab/Den Europæiske Union
DE Importeur in der Europäischen Gemeinschaft / Europäischen Union
EL Εισαγωγέας στην Ευρωπαϊκή Κοινότητα / Ευρωπαϊκή Ένωση
ES Importador en la Comunidad Europea / Unión Europea
FI Valtuutettu edustaja/vastuustahtenkilö Yhdystysneissä kuningaskunnassa
FR Importateur dans la Communauté européenne et l'Union européenne
IT Importatore nella Comunità europea/Unione Europea
NL Importeur in de Europese Gemeenschap / Europese Unie
NO Importør i Det europeiske fellesskap / Den europeiske union
PT Importador na Comunidade Europeia / União Europeia
SK Dovozca v Európskom spoločenstve alebo Európskej únii
SV Importör inom EU/EES
TR Avrupa Topluluğu / Avrupa Birliği/indeki ithalçısı
ZH 进口商欧洲共同体/欧盟的

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