
INTENDED USE: Percutaneous drainage of abscess, nephrostomy and other fluids. This product is intended for short-term use (less than 30 days).

INTENDED USERS: This product is intended only for use by trained clinicians.

**NON- LOCKING CATHETER:
GENERAL PURPOSE CATHETER WITH “J” TIP**

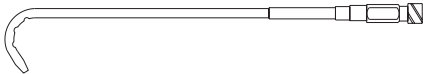**HARDWARE:
METAL STIFFENING CANNULA**

TROCAR

NOTE: Depth marks (if present) not intended to measure – for reference only

INSTRUCTIONS FOR USE:

1. To maximize advantages of the surface, WET THE CATHETER before use.
2. Straighten the catheter when inserting the cannula (and trocar if appropriate).
3. Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement. Catheters with a trocar included can also be inserted using direct puncture.
4. For pigtail catheters, rotate the catheter clockwise while removing the cannula to form the pigtail.
5. When the catheter is positioned, remove the hardware and fix the catheter in place.

CAUTION:

1. When engaging the catheter luer lock, do not overtighten. Do not use instruments to tighten connections. Overtightening the luer can result in luer crack which may require catheter replacement.
2. Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.

WARNING: The reuse of this single-use device can lead to patient infection and/or device malfunction.

SAFE DISPOSAL: Dispose of used device in container marked for biohazard (i.e. contaminated with potentially infectious substances of human origin).

REPORTING SERIOUS INCIDENTS: Any serious incident that has occurred in relation to the device should be reported to UreSil as the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

URESIL and Origin are registered trademarks of UreSil, LLC. GP is a trademark of UreSil, LLC.

REF

EN Catalog number
 CS Katalogové číslo
 DA Katalognummer
 DE Katalognummer
 EL Αριθμός καταλόγου
 ES Número de catálogo
 FR Numéro de catalogue
 IT Numero di catalogo
 NL Catalogusnummer
 NO Katalognummer
 PT Referência
 SK Katalógové číslo
 SV Katalognummer
 TR Katalog numarası
 ZH 产品目录号

LOT

EN Batch code
 CS Kód šarže
 DA Batchkode
 DE Chargencode
 EL Κωδικός παρτίδας
 ES Código de lote
 FR Code du lot
 IT Codice lotto
 NL Batchcode
 NO Partikode
 PT Código do lote
 SK Kód šarže
 SV Batchkod
 TR Parti kodu
 ZH 批号

EN Caution
 CS Upozornění
 DA Forsigtig
 DE Achtung
 EL Προσοχή
 ES Precaución
 FR Avertissement
 IT Attenzione
 NL Opgelet
 NO Advarsel
 PT Atenção
 SK Upozornenie
 SV Varning
 TR Dikkat
 ZH 注意

EN Country of manufacture/ Date of manufacture
 CS Země výroby/ Datum výroby
 DA Fremstillingsland/ Fremstillingsdato
 DE Herstellungsland/ Herstellungsdatum
 EL Χώρα κατασκευής/ Ημερομηνία κατασκευής
 ES País de fabricación/ Fecha de fabricación
 FR Pays de fabrication/ Date de fabrication
 IT Paese di produzione/ Data di produzione
 NL Land van productie/ Productiedatum
 NO Produksjonsland/ Produksjonsdato
 PT País de fabric/ Data de fabrico
 SK Krajina výroby/ Dátum výroby
 SV Tillverkningsland/ Üretim tarihi
 TR Ambalaj hasarlıysa ürünü kullanmayın ve kullanım talimatlarına başvurun/ Tillverkningsdatum
 ZH 产地/ 生产日期

EN Quantity
 CS Množství
 DA Antal
 DE Menge
 EL Ποσότητα
 ES Cantidad
 FR Quantité
 IT Quantità
 NL Hoeveelheid
 NO Antall
 PT Quantidade
 SK Množstvo
 SV Kvantitet
 TR Miktar
 ZH 数量

EN Not made with natural rubber latex
 CS Není vyrobeno z latexu z přírodního kaučuku
 DA Ikke fremstillet med naturgummilætex
 DE Nicht aus Naturkautschuklatex hergestellt
 EL Δεν κατασκευάζεται με λατέξ από φυσικό καουτσούκ
 ES No fabricado con látex de caucho natural
 FR Non fabriqué avec du latex de caoutchouc naturel
 IT Non realizzato con lattice di gomma naturale
 NL Niet gemaakt van natuurlijk rubberlatex
 NO Skal ikke brukes med naturlig gummlætex
 PT Não fabricado com látex de borracha natural
 SK Neobsahuje prírodný latex
 SV Inte tillverkad med naturlig gummlætex
 TR Doğal kauçuk lateksten yapılmamıştır
 ZH 并非天然乳胶材质

EN Do not re-use
 CS Nepoužívejte opakovaně
 DA Må ikke genbruges
 DE Nicht wiederverwenden
 EL Να μην επαναχρησιμοποιηθεί
 ES No reutilizar
 FR Ne pas réutiliser
 IT Non riutilizzare
 NL Niet hergebruiken
 NO Ikke til gjenbruk
 PT Não reutilizar
 SK Nepoužívajte opakovane
 SV Får inte återanvändas
 TR Yeniden kullanmayın
 ZH 不可重复使用

1-800-538-7374 (U.S.A.)

1-847-982-0106

EN Telephone Number
 CS Telefonní číslo
 DA Telefonnummer
 DE Telefonnummer
 EL Αριθμός τηλεφώνου
 ES Número de teléfono
 FR Numéro de téléphone
 IT Numero di telefono
 NL Telefoonnummer
 NO Telefonnummer
 PT Número de telefone
 SK Telefónne číslo
 SV Telefonnummer
 TR Telefon Numarası
 ZH 电话号码

EN Fax Number
 CS Číslo faxu
 DA Faxnummer
 DE Faxnummer
 EL Αριθμός φαξ
 ES Número de fax
 FR Numéro de fax
 IT Numero di fax
 NL Faxnummer
 NO Faksnummer
 PT Número de fax
 SK Faxové číslo
 SV Faxnummer
 TR Faks Numarası
 ZH 传真号码

EN Do not re-sterilize
 CS Neresterilizujte
 DA Må ikke resteriliseres
 DE Nicht erneut sterilisieren
 EL Να μην αποστειρωθεί εκ νέου
 ES No reesterilizar
 FR Ne pas restériliser
 IT Non risterilizzare
 NL Niet opnieuw steriliseren
 NO Skal ikke resteriliseres
 PT Não reesterilizar
 SK Nesterilizujte
 SV Får inte omsteriliseras
 TR Yeniden sterilize etmeyin
 ZH 不可重复消毒

CH REP

MedEnvoy Switzerland, Gotthardstrasse 28,
 6302 Zug Switzerland

EN Authorised Representative in Switzerland/Importer in Switzerland
 CS Oprávněný zástupce ve Švýcarsku / Dovožce ve Švýcarsku
 DA Autoriseret repræsentant i Schweiz/Importør i Schweiz
 DE Bevollmächtigter Vertreter in der Schweiz / Importeur in der Schweiz
 EL Εξουσιοδοτημένος εκπρόσωπος στην Ελλάδα/ εισαγωγέας στην Ελλάδα
 ES Representante autorizado en Suiza/Importador en Suiza
 FR Représentant autorisé en Suisse / Importateur en Suisse
 IT Rappresentante autorizzato in Svizzera/Importatore in Svizzera
 NL Geautoriseerde vertegenwoordiger in Zwitserland/importeur in Zwitserland
 NO Autorisert representant i Sveits / importør i Sveits
 PT Representante autorizado na Suíça/ Importador na Suíça
 SK Autorizovaný zástupca vo Švajčiarsku/dovožca vo Švajčiarsku
 SV Auktoriserad representant i Schweiz/importör i Schweiz
 TR İsviçre'deki Yetkili Temsilci/ İsviçre'deki İthalatçı
 ZH 瑞士授权代表/瑞士进口商

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