TCS-systembeskrivelse:**Funktioner/fordele:**

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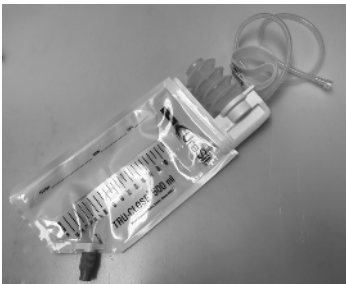
TCS System Description:

Features/Benefits:

- The TRU-CLOSE Suction Drainage System is designed to be a completely closed system for abscess and other fluid drainage.
- Compact and self-contained system allows full patient ambulation.
- During activation, body fluids are not aerosolized because all fluids are contained in the system.
- Closed system design reduces the chances of cross-contamination.
- Anti-reflux valve prevents liquids and air from backing up into the catheter or wound drain.
- All TCS bags come with a standard luer lock connector to fit all luer-locking drainage catheters.
- If the bellows fills, flow to the bag continues via gravity drainage as long as the bag is below the level of the drainage site.
- Bag sizes: 300, 500 and 1,000 ml.
- Available with empty port in the 300 ml and 500 ml sizes.


Technical Specifications

Catalog Number	Bag Size	Drain Port?	Luer Lock?
TCS300D	300 ml	Yes	Yes
TCS500DS	500 ml	Yes	Yes
TCS500D	500 ml	Yes	Yes
TCS500	500 ml	No	Yes
TCS1000	1000 ml	No	Yes

Patient and Nursing Information

- To adjust the tubing length, remove the tube from the top of the housing or pull off the luer lock connector and cut the tube with a scissors to the desired length (see Figures 1 and 2).
- Attach the bag to the drainage catheter or wound drain.
- To ensure continuous flow, attach the bag to the patient down or bedding below the level of the drainage site.
- If the bag comes with an empty port, twist it closed (see Figure 3).
- To activate the system, compress the bellows completely (see Figure 4). Do not try to pump the bellows; it will fill as it draws fluid from the drainage site. If the bellows is not re-activated after filling, the system will convert to gravity drainage thus preventing fluid backup.
- To re-activate the system, compress the bellows. The fluid in the bellows will be discharged into the bag and the suction will be re-established (see Figure 4).
- The bag is strong enough to withstand a build up of air pressure within the bag. The bag is equipped with hydrophobic filter vents, which will vent collected air. Air can be manually forced out of the bag by gently squeezing the bag while the system is positioned vertically (housing above bag).
- Although the graduation marks on the bag are approximations only, approximate volumes collected can be recorded on the white "write-on" area on the bag. When recording these approximations, first empty bellows contents into the bag.

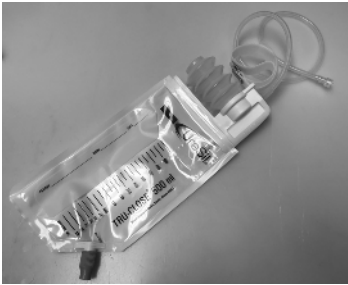
UreSil and TRU-CLOSE are registered trademarks of UreSil, LLC.

1

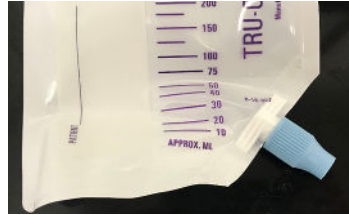2

3

4

UreSil, LLC,
5418 W. Toubay Ave, Skokie, IL
60077 U.S.A.

EN Manufacturer
BG Производител
CS Výrobce
DA Producent
DE Hersteller
EL Κατασκευαστής
ES Fabricante
FR Fabricant
IT Produttore
ES Número de teléfono
FR Numéro de téléphone
RO Producător
SK Výrobca
SV Tillverkare
TR Üretici
ZH 制造商

EC REP

EMERGO EUROPE, Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands

EN Authorized representative in the European Community / European Union

BG Упълномощен представител в Европейската общност / Европейския съюз
CS Zplnomocněný zástupce v Evropském společenství / Evropské unii
DA Autoriseret repræsentant i Det Europæiske Fællesskab/Den Europæiske Union
DE Bevollmächtigter in der Europäischen Gemeinschaft / Europäischen Union
EL Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Ευρωπαϊκή Ένωση
ES Representante autorizado en la Comunidad Europea / Unión Europea
FR Représentant autorisé dans la Communauté européenne et l'Union européenne
IT Rappresentante autorizzato nella Comunità europea/Unione Europea
NL Geautoriseerde vertegenwoordiger in de Europese Gemeenschap / Europese Unie
PT Representante autorizado na Comunidade Europeia / União Europeia
RO Représentant autorizat în Comunitatea Europeană / Uniunea Europeană
SK Zplnomocnený zástupca v Európskom spoločenstve alebo Európskej únii
SV Auktoriserad representant inom EU/EES
TR Avrupa Topluluğu / Avrupa Birliği'ndeki yetkili temsilci
ZH 欧洲共同体/欧盟的授权代表

MedEnvoy, Prinses Margrietplantsoen 33
Suite 123, 2595 AM, The Hague, The Netherlands

EN Importer in the European Community / European Union
BG Вносител в Европейската общност / Европейския съюз
CS Dovoze v Evropském společenství / Evropské unii
DA Importør i Det Europæiske Fællesskab/Den Europæiske Union
DE Importeur in der Europäischen Gemeinschaft / Europäischen Union
EL Εισαγωγέας στην Ευρωπαϊκή Κοινότητα / Ευρωπαϊκή Ένωση
ES Importador en la Comunidad Europea / Unión Europea
FR Importateur dans la Communauté européenne et l'Union européenne
IT Importatore nella Comunità europea/Unione Europea
NL Importeur in de Europese Gemeenschap / Europese Unie
PT Importador na Comunidade Europeia / União Europeia
RO Importator în Comunitatea Europeană / Uniunea Europeană
SK Dovožca v Európskom spoločenstve alebo Európskej únii
SV Importör inom EU/EES
TR Avrupa Topluluğu / Avrupa Birliği'ndeki ihalatçı
ZH 进口商/欧洲共同体/欧盟的

MD

EN Medical device
BG Медицинско изделие
CS Zdravotnický prostředek
DA Medicinsk udstyr
DE Medizinprodukt
EL Ιατροτεχνολογικό προϊόν
ES Producto sanitario
FR Dispositif médical
IT Dispositivo medico
NL Medisch apparaat
PT Dispositivo médico
RO Dispozitiv medical
SK Zdravotnícka pomôcka
SV Medicinsk utrustning
TR Tıbbi cihaz
ZH 医疗设备

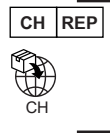
1-800-538-7374 (U.S.A.)

EN Telephone Number
BG Номер на факс
CS Telefonní číslo
DA Telefonnummer
DE Telefonnummer
EL Αριθμός τηλεφώνου
ES Número de teléfono
FR Numéro de téléphone
RO Producător
SK Výrobca
SV Tillverkare
TR Üretici
ZH 电话号码

1-847-982-0106

EN Fax Number
BG Номер на факс
CS Číslo faxu
DA Faxnummer
DE Faxnummer
EL Αριθμός φαξ
ES Número de fax
FR Numéro de fax
IT Numero di telefono
NL Telefoonnummer
PT Número de telefone
RO Număr de telefon
SK Telefónne číslo
SV Faxnummer
TR Telefon Numarası
ZH 传真号码

MedEnvoy Switzerland, Gotthardstrasse 28,
6302 Zug Switzerland

EN Authorised Representative in Switzerland/Importer in Switzerland
BG Упълномощен представител в Швейцария/ Вносител в Швейцария
CS Oprávněný zástupce ve Švýcarsku / Dovoze v Švýcarsku
DA Autoriseret repræsentant i Schweiz/Importer i Schweiz
DE Bevollmächtigter Vertreter in der Schweiz / Importeur in der Schweiz
EL Εξουσιοδοτημένος εκπρόσωπος στην Ελβετία/εισαγωγέας στην Ελβετία
ES Representante autorizado en Suiza/Importador en Suiza
FR Représentant autorisé en Suisse / Importateur en Suisse
IT Rappresentante autorizzato in Svizzera/Importatore in Svizzera
NL Geautoriseerde vertegenwoordiger in Zwitserland/Importeur in Zwitserland
PT Representante autorizado na Suíça/ Importador na Suíça
RO Représentant autorizat în Elveția / Importator în Elveția
SK Auktoriserad representant i Schweiz/Importör i Schweiz
SV Auktoriserad representant i Schweiz/Importör i Schweiz
TR İsviçre'deki Yetkili Temsilci/ İsviçre'deki İthalatçı
ZH 瑞士授权代表/瑞士进口商

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