

# Evolution® PRE-VAC WOUND DRAINAGE BOTTLE, 1000 ml



Refer to Label



Caution: Federal law (U.S.A.) restricts this device to sales by or on the order of a physician. Re-use of this product will cause the risk of cross infection and unpredictable health threat. This product is not made with natural rubber latex.

## Directions for use

1. Open the pouch.
2. Inspect the vacuum indicator is at "MAX" line. If not, do not use this product.
3. CLOSE CONNECTING TUBE CLAMP FIRST! Attach Male Luer Connector of the connecting tube (if provided) to Female Luer of patient drainage device ; if using a spike connector for drainage, wipe rubber stopper with disinfectant as per hospital protocol and then push spike through rubber stopper to begin drainage.
4. After checking all connections are secure, open the white slide clamp on the bottle to activate vacuum.
5. Open connecting tube clamp to begin drainage procedure.
6. Repeat steps 1 to 5 with new bottle if vacuum indicator is at "**MIN**" Line, or bottle is almost full and further drainage is required.
7. When fluid flow stops or the bottle is full, disconnect bottle from tubing set and ensure pinch clamp is engaged to prevent leaking.
8. To empty fluid from bottle, remove rubber stopper on pour spout and discard contents of bottle as per physician instructions and standard protocols for biohazard waste disposal.

**Manufactured By:** PACIFIC HOSPITAL SUPPLY CO., LTD  
4F., NO. 160, DAYE ROAD, BEITOU DISTRICT, TAIPEI 112, TAIWAN  
Factory: No. 8, Tongke 2<sup>nd</sup> Rd., Tongluo Township, Miaoli 366, Taiwan  
Tel : 886 2-2895-5050 Fax: 886 2-2897-8282  
E-mail : [pahsco@pahsco.com.tw](mailto:pahsco@pahsco.com.tw)  
Made In Taiwan

**Distributed by:** UreSil, LLC.  
5418 W. Touhy Ave.  
Skokie, IL 60077, USA



**Rx ONLY**



TO REORDER:  
Call: 1-800-538-7374  
Fax: 1-847-982-0106  
[www.uresil.com](http://www.uresil.com)

Evolution and UreSil are registered trademarks of UreSil, LLC

Products Covered:

**ESB-1001SP** PRE-VAC DRAINAGE BOTTLE 1000 ML TUBING NON-STERILE

**ESB-1001FS-T** PREVAC STERILE WOUND DRAINAGE BOTTLE 1000 ML W/TUBING